


AFIAS

NT-proBNP

This is not a complete Instruction for use. For more detailed instructions, please refer to IFU.

Test Components

Check the components of the AFIAS as described below.

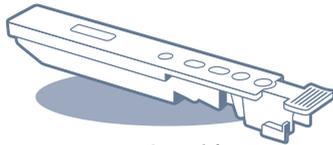

Test Cartridge



Tip




ID Chip




Ensure that the lot number of the cartridge matches that on the ID chip.

LOT Matches **LOT**

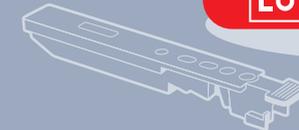

Test Setup

- 1 Leave the sealed cartridge (if stored in the refrigerator) at room temperature for at least 30 minutes before use.

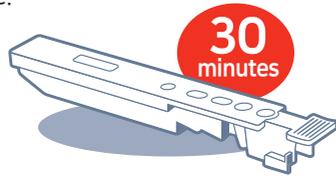


Place the cartridge on a clean, dust-free and flat surface.


- 2 Turn on the power. Flip the main power switch to "I", located on the back of the reader.



- 3 Press the power button, located on the front panel.

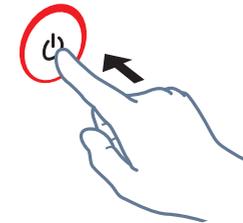


- 4 Empty the tip box and tap "Reset".

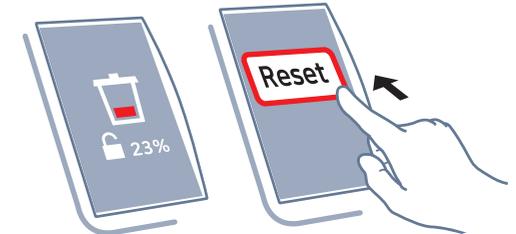

Test Procedure


- 5 Insert the ID chip into the "ID chip port".

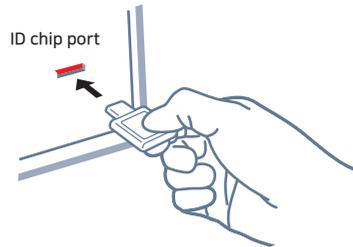


- 6 Tap the "General mode" icon on the screen.



- 1 Insert the cartridge fully into the cartridge holder.

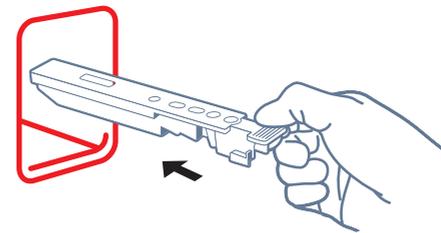


- 2 Insert a tip into the tip hole of the cartridge.

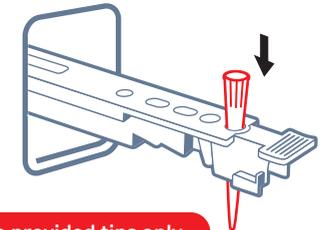

Use provided tips only.


- 3 Take 100 µL of sample with a pipette and dispense it into the sample well on the cartridge.

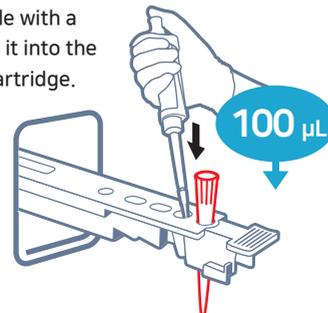

WB, Serum, Plasma



- 4 To start the test, tap the 'START' icon on the screen.

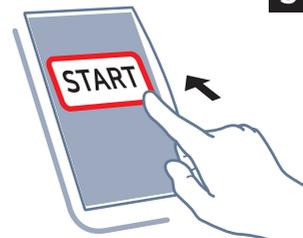

- 5 The test result will be displayed on the screen in 12 minutes.

12 minutes

